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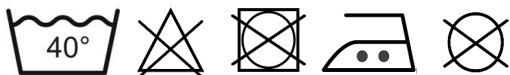
Directions for use

Put the apron over your head and tie the straps behind your back. If the user does not wish to tie the straps they can be ties together and placed under the apron in the lap of the user. Make sure the long straps do not hang down so that the user stumbles upon them if they stand up.

General Safety Precautions

- The Komikapp activity apron may not be used by persons who are disoriented, without supervision.
- The long sidebands increase the risk of falling or getting caught somewhere.
- There is a danger of strangulation caused by the adjustable neckband.
- The activity apron includes small parts and must not be used by children under the age of 3 without supervision.
- Patients who cannot remove the apron themselves must always be under supervision.
- Think of unsnapping all ribbons or strings if the patient has a tendency to cause injury to them.
- The activity apron has long side straps which may only be tied around the body/waist of the patient. Please be aware that the Komikapp activity apron is not intended to restrict a person's freedom of movement or to fix them for safety reasons.

Duration of application

The duration of application depends on the patient's need for stimulation. Tip; change hangers/objects to according to the user's needs, for example a bunch of keys, exciting key rings, something that makes a sound...

Care instructions

The activity apron is made of 100% cotton and washable up to 40° incl. muff.

Remove loose / attached materials and objects before washing and clean separately or disinfect.