**Protac Kugledynen™**

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**Protac Ball Blanket™**

GB - Directions for use

**Protac Kugeldecke™**

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**Protac pallopeitto™**

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**Protac Ball Blanket™**

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**Protac Ball Blanket™**

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**Protac Bolltäcke™**

SE - Bruksanvisning

**Protac Ball Blanket™**

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**Protac Ball Blanket™**

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Medical device

[www.protac.dk](http://www.protac.dk)

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Protac Kugledynen™ Classic

Protac Kugledynen™ Flexible

Protac Kugledynen™ Calm

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Protac Ball Blanket™ Flexible

Protac Ball Blanket™ Calm

You have selected a quality Protac product. Protac guarantees that you have purchased an original, Danish designed product developed in consultation with experts.

Protac has developed, produced and sold sensorystimulating aids to the healthcare sector in Denmark, the rest of Europe, Australia and New Zealand since 1994.

Protac products are developed on the basis of theories on sensory integration, and are supported by extensive research, documentation and case studies.

Protac focuses strongly on protection of the environment and safety during its production process. Protac plastic balls are TÜV tested and complies with REACH. The textiles are Oeko-Tex 100 certified.

For more details, go to [www.protac.dk](http://www.protac.dk)

GB

## Labelling

Our products are labelled in accordance with legislation for medical devices class I.

The label with details of manufacturer, raw materials and washing instructions is sewn into the foot of the blanket to avoid irritating the user.

Key to label symbols: please refer to **Key to symbols**

## General precautions - Safety

- ⚠ • **Not for unsupervised use** by users unable to care for themselves.
- ⚠ • **Not for unsupervised use** by users with low lung capacity.
- ⚠ • **Not to** be used as a means of restraint.
- ⚠ • Please **pay attention** to the reaction of the user, if he or she does not like lying under the quilt.
- ⚠ • When the Protac Ball Blanket™ is used with an incontinence cover, it is important to be aware of the possible risks.  
**The same risk precautions apply as for the use of a plastic bag:**
  - Incontinence covers are **dangerous toys for children**.
  - A Protac Ball Blanket™ used with an incontinence cover **must not** be used for children without supervision.
  - A Protac Ball Blanket™ used with an incontinence cover **must not** be used for patients unable to care for themselves **without supervision**.
  - A Protac Ball Blanket™ used with an incontinence cover **must not** be pulled over the head.
- ⚠ • Protac Ball Blanket™ with polystyrene granulate contain small particles, and **must not** be used for children below the age of 3 **without supervision**.
- ⚠ • Smoking is forbidden under the Protac Ball Blanket™. Embers from cigarettes can cause burns.
- ⚠ • Never leave a computer or other electrical appliances switched on in a bed with a Protac Ball Blanket™, as they emit heat which can be harmful to the blanket, or in extreme circumstances, cause the appliance to ignite, due to a build-up of heat.
- ⚠ • **Must not** be used near an open fire.

## Product purpose and areas of use

Sensory integration is the idea behind the development of all Protac products. The theory originated in the 1970s in the USA, and was developed by A. Jean Ayres, an occupational therapist and psychologist. The theory is based on how the brain processes sensory impressions, and how the various sensory stimuli can be influenced via applying weight and pressure to the tactile sense (the sense of touch) in the skin and the proprioceptive sense (the sense of movement) in muscles and joints. The pressure, movement and weight of the balls act on these senses in particular.

The Protac Ball Blanket™ was developed to help sufferers of function impairment.

Children, young and old people can benefit from the Protac Ball Blanket™:

- Brain damage, including spasticity, Huntington's Chorea, Parkinson's Disease and other ailments
- ADHD and autism
- Psychiatric diseases, such as dementia, schizophrenia, depression, anxiety, anorexia etc.
- Development problems, sensory problems, deaf/blind, etc.
- Stress

GB

The Protac Ball Blanket™ can be used for patients with brain damage resulting in spasticity. The weight and pressure of the balls help suppress spasticity, and calm the body of patients with brain damage. Furthermore, the patient benefits from enhanced body awareness.

The Protac Ball Blanket™ is used by ADHD and autism sufferers to sleep with, and as a source of comfort during the daytime. The pressure and weight of the balls impart sensory stimuli which help define the body, giving comfort and calm. ADHD sufferers often have a problem winding down at night, and the Protac Ball Blanket™ gives the body a signal to relax through its pressure and weight.

Psychiatric patients often suffer from muscular and psychological unrest. The Protac Ball Blanket™ is used to create calm and comfort for the user. The weight and pressure of the balls also give a better feeling of body awareness and enhanced wellness. The Protac Ball Blanket™ is used to sleep with, and to provide comfort during the daytime.

The Protac Ball Blanket™ is used for sufferers of a range of development problems, sensory problems, deaf/blindness and to create enhanced sensory stimuli in the body.

The Protac Ball Blanket™ is used for sufferers of stress to provide calmness, comfort and better body awareness. It is used to sleep with, and to provide comfort during the daytime. Sufferers of stress often have a problem feeling their body, preventing them falling asleep at night. The pressure and weight of the Protac Ball Blanket™ give the body a signal to relax.

The Protac Ball Blanket™ can be used in institutions, sheltered housing, day centres, hospitals, patient lounges and private homes. The blanket can be used in schools in classrooms or association areas, for instance. At home, the blanket can be used in a bedroom or the lounge.

## Using the product

The user must enclose the Protac Ball Blanket™ in a normal quilt cover.

The Protac Ball Blanket™ is used in the same way as an ordinary duvet for sleeping. Pack it tightly around the body so that you can clearly feel the balls.

We recommend sitting with the blanket over you in a chair or sofa initially, to feel the weight of the balls, which will be a new experience.

For extra stimulation, place your hands on the Protac Ball Blanket™ and press the balls against your body.

You can also lie on top of the Protac Ball Blanket™. This will provide sensory stimulation to your back.

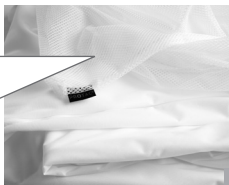
If the Protac Ball Blanket™ is to be used by a child, we recommend allowing the child to play with it first while awake by crawling under it, or sitting with the blanket over his/her legs.

## Accessories

### Protac Incontinence Cover

## Using the product

The Protac Incontinence Cover is used over the outside of the blanket if the patient is incontinent (involuntary urination). An ordinary duvet cover should be used over the incontinence cover for comfort (see the Protac Incontinence Cover directions for use).

GB

## General precautions - Safety

- ⚠ • We recommend that the opening of the incontinence cover is at the opposite end to that of the duvet cover (not applicable for incontinence cover with mesh top)
- ⚠ • **The same risk precautions apply as for the use of a plastic bag:**
  - Incontinence covers are **dangerous toys for children**.
  - A blanket used with an incontinence cover **must not** be used for children without supervision.
  - A blanket with incontinence cover **must not** be used for patients unable to care for themselves without supervision.
  - A blanket used with an incontinence cover **must not** be pulled over the head.
- ⚠ • We recommend using the incontinence cover version with a mesh top (breathable material) for users unable to care for themselves.

## Product service life

The product has an expected service life of 5 years.

This is based on correct use, cleaning, washing, service and maintenance.

Products used beyond the aforementioned service life should be assessed for further use.

Responsibility for using the product beyond the stated service life rests with the user.

## Cleaning

All Protac Ball Blanket™'s can be washed. The regularity and need for washing depends on use. See below for washing and cleaning instructions.

### Protac Ball Blanket™ Classic

- Item nos. 100-101, 100-102, 100-105, 100-106, can be machine washed in a min. 16 kg capacity washing machine with washing detergent with no bleaching agents - see also the washing instructions on the Protac Ball Blanket™.
- Item nos. 100-107 baby can be washed in a min. 7 kg capacity washing machine with washing detergent with no bleaching agents - see also the washing instructions in the Protac Ball Blanket™.

### **Protac Ball Blanket™ Flexible**

- Item nos. 100-201, 100-205, 100-202, 100-206 can be machine washed in a min. 7 kg washing machine with washing detergent with no bleaching agents. Remove the bags from the cover. The laundrybag delivered with 100-205 and 100-206 must be used when washing the bags. Wash cover and bags separately. Spinning at low speed - see also the washing instructions in the cover.

### **Protac Ball Blanket™ Calm**

- Item nos. 100-321 and 100-322 should be split in two and each washed in a min. 7 kg capacity washing machine with washing detergent with no bleaching agents. The enclosed laundrybag must be used. Spinning at low speed - see also the washing instructions in the Protac Ball Blanket™.

**GB**

To remove spots using a spot remover, always try first on an area which is not visible to test for damage or discolouring.

Protac does not recommend the use of spot remover.

Protac cannot be held liable for incorrect washing and maintenance.

## **Maintenance**

We recommend hanging the Protac Ball Blanket™ out to air on a regular basis.

## **Transport and storage**

The Protac Ball Blanket™ is packed in a carrying case and in a protective plastic bag that protects the blanket from dirt and moisture. The blanket is ready for use.

The Protac Ball Blanket™ can advantageously be transported in the carrying case so that it is easy to handle. If the blanket is not used, it must be stored securely in the carrying case so that it is protected from dirt and moisture.

## **Disposal**

The Protac Ball Blanket™ can be broken down into the following components for disposal: Plastic and textile.

The plastic balls can be delivered for recycling.

All components can be disposed of as combustible refuse.

Always dispose of refuse according to local rules.

## **Guarantee**

2 year guarantee against manufacturing defects.

## **The guarantee will be invalid in the event of the following:**

- overloading
- misuse
- non-maintenance or incorrect maintenance

**Traceability:** Production month, year and batch number are marked on the washing label.

## Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product

**Washing bowl:** Maximum washing temperature (see care label)

**Warning:**  
See under **Safety**

**Triangle:**  
Do not bleach

User information

**Square with a circle:**  
tumble drying possible  
(see care label)

For indoor use

**Iron:**  
Do not iron

Manufacturer

**Circle:**  
Do not dry clean

Batch number

GB

## Accessories

- Protac Incontinence cover
- Protac Bedlinen
- Protac Cover with thermolining
- Protac Carrying case

## Parts List

- Cover for Flexible blankets
- Ball bags for Flexible blankets
- Laundry bag

## Technical specifications

### Dimensions:

<b>Protac Ball Blanket™, Adult</b>	<b>length:</b> 200/220 cm	<b>width:</b> 140 cm
<b>Protac Ball Blanket™, Junior:</b>	<b>length:</b> 140 cm	<b>width:</b> 100 cm
<b>Protac Ball Blanket™, Baby:</b>	<b>length:</b> 100 cm	<b>width:</b> 70 cm
<b>Protac Ball Blanket™, Premature:</b>	<b>length:</b> 67 cm	<b>width:</b> 50 cm

**Standards:** Comply with general requirements for aids acc. to the following harmonized standards:

**Clinical evaluation:** EN 12182:2012

**Risk analysis:** DS/EN ISO 14971 : 2012

**Patent:** Protac Ball Blanket™ Calm is registered as a Utility Model - Patented.

Protac Ball Blanket™ Flexible is registered as a Utility Model - Patented.

**Fire test:** ISO 12952-1-2 : - applies to Protac Ball Blanket™ Calm made of Trevira CS  
ISO 12952-1 : 2010 applies to Protac Ball Blanket™ Flexible made of Trevira CS.

## Service information

No other balls can be used in a Protac Ball Blanket™ than those used by the manufacturer.

## Guide

- to daily inspection of the blanket before use

No.	Check	Remarks
1	<ul style="list-style-type: none"><li>All parts are in place, no loose parts</li></ul>	
2	<ul style="list-style-type: none"><li>No sign of holes, damage to the textile or seams</li></ul>	

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### Protac A/S