Protac Specialdyner

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Protac Special Blankets

GB - Directions for use

Protac Special Blankets

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Protac Erikoispeitto

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Protac Special Blankets

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Protac Special Blankets

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Protac Speciella Bolltäck

SE - Bruksanvisning

Protac Special Blankets

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Protac Special Blankets

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Protac Polystyrendynen™

Protac Polystyrendynen™ Flexible

Protac Granulatdynen™

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Protac Polystyrene Blanket™ Flexible

Protac Granulate Blanket™

You have selected a quality Protac product. Protac guarantees that you have purchased an original, Danish designed product developed in consultation with experts.

Protac has developed, produced and sold sensory-stimulating aids to the healthcare sector in Denmark, the rest of Europe, Australia and New Zealand since 1994.

Protac products are developed on the basis of theories on sensory integration, and are supported by extensive research, documentation and case studies.

Protac focuses strongly on protection of the environment and safety during its production process. The polypropylene granulate complies with REACH. The textiles are Oeko-Tex standard 100 certified.

For more details, go to www.protac.dk

GB

Labelling

Our products are labelled in accordance with legislation for medical devices class I.

The label with details of manufacturer, raw materials and washing instructions is sewn into the foot of the blanket to avoid irritating the user.

Key to label symbols: please refer to **Key to symbols**

General precautions - Safety

- ⚠ • **Not for unsupervised use** by users unable to care for themselves.
- ⚠ • **Not for unsupervised use** by users with low lung capacity.
- ⚠ • **Not to** be used as a means of restraint.
- ⚠ • Please **pay attention** to the reaction of the user, if he or she does not like lying under the quilt.
- ⚠ • When the Protac Special Blankets are used with an incontinence cover, it is important to be aware of the possible risks.
The same risk precautions apply as for the use of a plastic bag:
 - Incontinence covers are **dangerous toys** for children.
 - A Protac Special Blanket, used with an incontinence cover **must not** be used for children with out supervision.
 - A Protac Special Blanket, used with an incontinence cover **must not** be used for patients unable to care for themselves **without supervision**.
 - A Protac Special Blanket, used with an incontinence cover **must not** be pulled over the head.
- ⚠ • Protac Special Blankets, with polystyrene/polypropylene granulate contain small particles, and **must not** be used for children below the age of 3 **without supervision**.
- ⚠ • Smoking is forbidden under the Protac Special Blankets. Embers from cigarettes can cause burns.
- ⚠ • Never leave a computer or other electrical appliances switched on in a bed with a Protac Special Blanket, as they emit heat which can be harmful to the Protac Special Blankets, or in extreme circumstances, cause the appliance to ignite, due to a build-up of heat.
- ⚠ • **Must not** be used near an open fire.

Product purpose and areas of use

Sensory integration is the idea behind the development of all Protac products. The theory originated in the 1970s in the USA, and was developed by A. Jean Ayres, an occupational therapist and psychologist. The theory is based on how the brain processes sensory impressions, and how the various sensory stimuli can be influenced via applying weight and pressure to the tactile sense (the sense of touch) in the skin and the proprioceptive sense (the sense of movement) in muscles and joints. The pressure, movement and weight of the balls act on these senses in particular.

The Protac Special Blankets were developed to help sufferers of function impairment.

Children, young and old people can benefit from the Protac Special Blankets:

- Brain damage, including spasticity, Huntington's Chorea, Parkinson's Disease and other ailments
- ADHD and autism
- Psychiatric diseases, such as dementia, schizophrenia, depression, anxiety, anorexia etc.
- Development problems, sensory problems, deaf/blind, etc.
- Stress

GB

The Protac Special Blankets can be used for patients with brain damage resulting in spasticity. The weight of the blanket helps suppress spasticity, and calm the body of patients with brain damage. Furthermore, the patient benefits from enhanced body awareness.

The Protac Special Blankets are used by ADHD and autism sufferers to sleep with, and as a source of comfort during the daytime. The pressure of the blanket imparts sensory stimuli which helps define the body, giving comfort and calm. ADHD sufferers often have a problem winding down at night, and the Protac Special Blankets gives the body a signal to relax through its weight.

Psychiatric patients often suffer from muscular and psychological unrest. The Protac Special Blankets are used to create calm and comfort for the user. The weight of the balls also gives a better feeling of body awareness and enhanced wellness. Protac Special Blankets are used to sleep with, and to provide comfort during the daytime.

The Protac Special Blankets are used for sufferers of a range of development problems, sensory problems, deaf/blindness and to create enhanced sensory stimuli in the body.

The Protac Special Blankets are used for sufferers of stress to provide calmness, comfort and better body awareness. It is used to sleep with, and to provide comfort during the daytime. Sufferers of stress often have a problem feeling their body, preventing them falling asleep at night. The pressure and weight of the Protac Special Blanket gives the body a signal to relax.

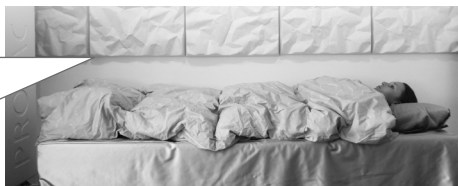
The Protac Special Blankets can be used in institutions, sheltered housing, day centres, hospitals, patient lounges and private homes. The Protac Special Blankets can be used in schools in classrooms or association areas, for instance. At home, the Protac Special Blankets can be used in a bedroom or the lounge.

Using the product

The user must enclose the Protac Special Blankets in a normal quilt cover.

The Protac Special Blankets are used in the same way as an ordinary duvet for sleeping. Pack it tightly around the body so that you can clearly feel the weight.

We recommend sitting with the Protac Special Blankets over you on a chair or sofa initially, to feel the weight of the blanket, which will be a new experience.

For extra stimulation, place your hands on the Protac Special Blankets and press the blanket against your body.

You can also lie on top of the Protac Special Blankets. This will provide sensory stimulation to your back.

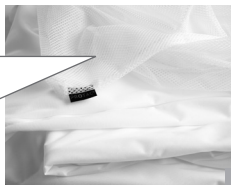
If the Protac Special Blankets is to be used by a child, we recommend allowing the child to play with it first while awake by crawling under it, or sitting with the blanket over his/her legs.

Accessories

Protac Incontinence Cover

Using the product

The Protac Incontinence Cover is used over the outside of the blanket if the patient is incontinent (involuntary urination). An ordinary duvet cover should be used over the incontinence cover for comfort (see the Protac Incontinence Cover directions for use).

GB

General precautions - Safety

- ⚠ • We recommend that the opening of the incontinence cover is at the opposite end to that of the duvet cover (not applicable for incontinence cover with mesh top)
- ⚠ • **The same risk precautions apply as for the use of a plastic bag:**
 - Incontinence covers are **dangerous toys** for children.
 - A blanket used with an incontinence cover **must not** be used for children without supervision.
 - A blanket with incontinence cover **must not** be used for patients unable to care for themselves without supervision.
 - A blanket used with an incontinence cover **must not** be pulled over the head.
- ⚠ • We recommend using the incontinence cover version with a mesh top (breathable material) for users unable to care for themselves.

Product service life

The product has an expected service life of 5 years.

This is based on correct use, cleaning, washing, service and maintenance.

Products used beyond the aforementioned service life should be assessed for further use. Responsibility for using the product beyond the stated service life rests with the user.

Cleaning

All Protac Special Blankets can be washed. The regularity and need for washing depends on use. See below for washing and cleaning instructions.

Protac Polystyren Blanket™

- Item nos. 100-109, 100-110 can be machine washed in a min. 16 kg capacity washing machine. Use washing powder without bleaching agents. Spinning at low speed - see also the washing instructions on the blanket.

Protac Polystyrene Blanket™ Flexible

- Item nos. 100-209, 100-210 can be machine washed in a min. 7 kg washing machine. Use washing powder without bleaching agents. Remove the bags from the cover. Wash cover and bags separately. Use the enclosed laundrybag. Spinning at low speed - see also the washing instructions on the cover.

Protac Granulate Blanket™

Item nos. 100-419, 100-429 with polypropylene granulate must be split in two and each part washed separately in a min. 7 kg capacity washing machine. Use washing powder without bleaching agents. Use the enclosed laundrybag. Spinning at low speed - see also the washing instructions in the blanket.

To remove spots using a spot remover, always try first on an area which is not visible to test for damage or discolouring.

Protac does not recommend the use of spot remover.

Protac cannot be held liable for incorrect washing and maintenance.

GB Maintenance

We recommend hanging the Protac Special Blankets out to air on a regular basis.

Transport and storage

The Protac Special Blanket is supplied in a handy carrying case.

The Protac Special Blanket is ready for use.

When transporting, pack the Protac Special Blanket in its case and a sheet of plastic to protect it from damp and dirt.

If you are not using the Protac Special Blanket, store it carefully in its case to protect from damp and dirt.

Disposal

The Protac Special Blankets can be broken down into the following components for disposal: Plastic and textile.

All components can be disposed of as combustible refuse.

Always dispose of refuse according to local rules.

Guarantee

2 year guarantee against manufacturing defects.

The guarantee will be invalid in the event of the following:

- overloading
- misuse
- non-maintenance or incorrect maintenance

Traceability: Production month, year and batch number are printed on the CE - and washing label.

Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product

Washing bowl: Maximum washing temperature (see care label)

Warning:
See under **Safety**

Triangle:
Do not bleach

User information

Square with a circle:
tumble drying possible
(see care label)

For indoor use

Iron:
Do not iron

CE approval according to Directive concerning medical devices 93/42/EC Class 1

Circle:
Do not dry clean

Manufacturer

Batch number

GB

Accessories

- Protac Incontinence cover
- Protac Bedding
- Protac Cover with thermolining
- Protac Carrying case

Spare parts

- Cover for Flexible
- Ballbags for Flexible
- Laundry bag

Technical specifications

Dimensions:

Protac Special Blankets

length: 200/220 cm

width: 140 cm

Standards: Comply with general requirements for aids acc. to the following harmonized standards:

Clinical evaluation: EN 12182:2012

Risk analysis: DS/EN ISO 14971:2012

Fire test: ISO 12952, 1 : 2010 - applies only to blankets made of Trevira CS

Service information

No other Granulate can be used in a Protac Special Blankets than those used by the manufacturer.

Guide

- to daily inspection of the blanket before use

No.	Check	Remarks
1	<ul style="list-style-type: none">All parts are in place, no loose parts	
2	<ul style="list-style-type: none">No sign of holes, damage to the textile or seams	

GB

Protac A/S

Niels Bohrs Vej 31 D, Stilling

DK-8660 Skanderborg

Tel.: 86 19 41 03

e-mail: protac@protac.dk

Dealer list:

Please refer to: www.protac.dk/kontakt

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