Protac KneedMe® To Go

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GB - Directions for use

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SE - Bruksanvisning

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Protac KneedMe® To Go

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GB - Directions for use

Protac KneedMe® To Go

You have chosen a Protac quality product. Protac guarantees that you have acquired an original, Danish design product, developed in cooperation with professional business partners. Since 1994, Protac has been designing, producing and selling sensory-stimulating devices for the health sector in Denmark, Europe and Australia.

Protac's products are developed on the basis of theories about sensory integration and are well-evidenced in terms of research, documentation and case studies. In the manufacturing process, Protac focuses on environmental and safety issues. Protac's plastic balls are TÜV-approved.

For further information see www.protac.dk

GB

Labelling

Products are labelled in compliance with regulations for medical equipment. A label with information about manufacturer, basic materials and washing instructions is attached inside the cover to protect the user from discomfort, when she/he is sitting using the knee blanket. The position also protects the label against wear and tear.

Key to label symbols: please refer to **Key to symbols**

General precautions - Safety

Please note that Protac KneedMe® To Go is not intended to curtail personal freedom or to be a safety device.

- ⚠ • The shoulder strap of the knee blanket can be used as a belt. This **must only** be fastened around the waist of the user, and **never** around the back of a chair. The belt prevents the knee blanket from slipping down when being used. When the strap has been adjusted and tightened correctly, it should fit closely around the user, comfortably and without being too tight.
- ⚠ • Users **who are unable to** unfasten the belt and free themselves from the knee blanket without assistance, should always be supervised.
- ⚠ • If a user's movements or reduced cognitive abilities inadvertently lead to release from the knee blanket, **we recommend** that the user should also be supervised.
- ⚠ • **Do not** use near an open flame.
- ⚠ • **Avoid** tripping over the shoulder strap/belt.
- ⚠ • When using Protac KneedMe® in a car, **it must not be used** by the person driving the car.
- ⚠ • If a passenger is using Protac KneedMe®, **it must** lie loosely over the knees. **Do not** use the shoulder strap around your waist as a belt.

The product's purpose and application environment

Sensory integration is the basis for the design of all Protac's products. This theory originated in the United States and was developed by A. Jean Ayres, an occupational therapist and psychologist, back in the 1970s. The theory is based on how the brain processes sensory impressions, and how one can influence the various sensory stimuli via weight and tactile sense (the sense of touch) in the skin and the proprioceptive sense (sense of movement) in the muscles and joints. The pressure, movement and weight of the balls have an impact on these senses.

Protac KneedMe® To Go is designed as an aid for people with disabilities.

Children from the age of 5/6 can use Protac KneedMe® To Go:

- Motoric and mental unrest
- Difficulty concentrating
- Sensory disturbance
- ADHD
- Autism
- Developmental irregularities

Protac KneedMe® To Go is ideal for users with motoric and mental unrest. Many children and young people have problems sensing themselves, but when they have one of these knee blankets on, they get an excellent awareness of their body. They calm down and concentrate.

Protac KneedMe® To Go can be used in institutions, residences, day care facilities, hospitals, common rooms and private rooms. In the home, Protac KneedMe® To Go can be used, for example, in the user's bedroom and in the living room.

Use of the product

Protac KneedMe® To Go should lie over the user's lap.

Attach the shoulder strap by inserting the snap hook through the first ring and securing it in the second ring.

You can fold up the Protac KneedMe® To Go and carry it by the two handles.

If Protac KneedMe® To Go is slipping off during use, you can use the shoulder strap as a belt to fasten round the waist.

You can fold up the Protac KneedMe® To Go and carry it using the shoulder strap.

If you have a tablet or mobile phone in your Protac KneedMe® To Go, it is a good idea to zip it into the rubber outer section for better protection during transport.

The product's service life

The product has an expected service life of 5 years.

which are used for longer than the above-mentioned service life, should be assessed prior to further use. It is the responsibility of the user to check regularly whether the product can be used for longer than the stipulated service life.

Cleaning

Protac KneedMe® To Go is washable. How frequently it needs washing depends on use. When washing and cleaning, please follow the instructions below.

Protac KneedMe® To Go

- Protac KneedMe® To Go can be washed in a washing machine with a min. capacity of 6 kg, using washing powder without bleach. Spinning at low speed. Also see the washing instructions in the cover.
- The cover and ball bag should be washed separately. For the ball bag, we advise you to use the washing bag that comes with the product.

GB

For removing stains, when you are considering using a stain remover, first try it on a place that is not visible, to see whether it might damage the fabric or affect the colour. Protac does not recommend the use of stain removers.

Protac reserves the right not to replace a product that has been wrongly washed and maintained.

Maintenance

We recommend airing the knee blanket regularly. Check your Protac KneedMe® To Go every time you use it for flaws and defects, and once a year inspect the knee blanket's seams, zip, fabric, plastic balls and general condition. If you become aware of a serious flaw or defect, stop using your Protac KneedMe® To Go until the flaw has been remedied and safety restored.

Transport and storage

Protac KneedMe® To Go comes ready to use.

During transport, the knee blanket is packed in plastic to protect it against dirt and moisture. When the knee blanket is not being used, it should be stored safely to protect it against dirt and moisture.

Disposal

When disposing of a Protac KneedMe® To Go, it can be divided into the following parts: plastic and fabric.

All parts can be disposed of as combustible waste.

Waste should be disposed of in accordance with applicable regulations.

Warranty

There is a 2-year warranty for manufacturing defects

The warranty is void in the following instances:

- Overload of the Protac KneedMe® To Go
- Heavy-handed use of the Protac KneedMe® To Go
- Lack of, or improper maintenance of the Protac KneedMe® To Go

Traceability: the month and year of manufacture are marked on the back of the washing label

Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product

Washing bowl: Maximum washing temperature (see care label)

Warning:
See under **Safety**

Triangle:
Do not bleach

User information

Square with a circle:
tumble drying possible
(see care label)

For indoor use

CE approval according to Directive concerning medical devices 93/42/EC Class I

Iron:
Do not iron

Manufacturer

Circle:
Do not dry clean

Product serial number (see care label)
700-755

GB

Technical specifications

Standards: Comply with general requirements for aids acc. to the following harmonized standards:

Clinical evaluation: EN 12182:2012

Risk analysis: DS/EN ISO 14971

Service information

Do not use balls in your Protac KneedMe® To Go other than those used by the manufacturer.

Guide

- to daily inspection Protac KneedMe® To Go prior to use

No.	Check	Remarks
1	<ul style="list-style-type: none">All parts are in place, no loose parts	
2	<ul style="list-style-type: none">No evidence of holes, or damage to fabric and stitching	

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**Protac A/S**