## Protac MyFit®

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GB - Directions for use

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SE - Bruksanvisning

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Medical device

[www.protac.dk](http://www.protac.dk)

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Protac MyFit®

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# GB - Directions for use

Protac MyFit®

You have selected a quality Protac product. Protac guarantees that you have purchased an original, Danish designed product developed in consultation with experts.

Since 1994, Protac has developed, manufactured and sold sensory stimulating aids to the health-care sectors in Denmark, Europe, Australia and New Zealand.

Protac products are developed on the basis of theories on sensory integration, and are supported by extensive research, documentation and case studies.

Protac focuses strongly on protection of the environment and safety during its production process. Protac plastic balls are TÜV tested and complies with REACH.

For more details, go to [www.protac.dk](http://www.protac.dk)

GB

## Labelling

Our products are labelled in accordance with to legislation for medical devices class I. The label is placed inside the vest to avoid irritating the user when wearing the garment.

Key to label symbols: please refer to **Key to symbols**

## General precautions - Safety

- ⚠ • Please **be aware** of how the user reacts.
- ⚠ • The Protac MyFit® vest **must not** be used as a swimming aid or life vest.
- ⚠ • **Not for unsupervised use** by users with a poor sense of balance.
- ⚠ • **Not for unsupervised use** by users unable to care for themselves.
- ⚠ • **Do not** use near open fire.
- ⚠ • **Must not** be used while sleeping.
- ⚠ • **Must not** be used by driver nor passengers when driving in a car
- ⚠ • The vest **must** be removed before the Heimlich method is performed

## Product purpose and areas of use

Sensory integration is the idea behind the development of all Protac products. The theory originated in the 1970s in the USA, and was developed by A. Jean Ayres, an occupational therapist and psychologist. The theory is based on how the brain processes sensory impressions, and how the various sensory stimuli can be influenced via applying weight and pressure to the tactile sense (the sense of touch) in the skin and the proprioceptive sense (the sense of movement) in muscles and joints. The pressure, movement and weight of the balls act on these senses in particular.

The Protac MyFit® was developed to help sufferers of function impairment.

Children, young and old people can benefit from the vest:

- Brain damage, including spasticity, Huntington's Chorea, Parkinson's Disease and other ailments
- ADHD and autism
- Psychiatric diseases, such as dementia, schizophrenia, depression, anxiety, anorexia etc.
- Development problems, sensory problems, deaf/blind, etc.
- Stress

The Protac MyFit® is used for treatment of the after-effects suffered by brain damage patients, to strengthen balance and body awareness.

It is used for ADHD and autism sufferers to help with concentration and learning difficulties, as the thickness and pressure of the balls give extra sensory stimuli and better body awareness.

Psychiatric patients often suffer from muscular and psychological unrest. The Protac MyFit® vest is used to create calm and comfort for the user. The weight and pressure of the balls also give a better feeling of body awareness and enhanced wellness.

It is used for sufferers of a range of development problems, sensory problems, deaf/blindness and to create enhanced sensory stimuli in the body.

It is used for sufferers of stress to provide calmness, comfort and better body awareness.

The Protac MyFit® can be used in institutions, sheltered housing, day centres, hospitals, patient lounges and private homes. Schoolchildren can wear the vest in the classroom or playtimes.

At home, the vest can be worn at mealtimes, when doing homework or whenever extra concentration is needed.

## Using the product

1. Put on and zip up the Protac MyFit® vest.

2. Model 800-850-10 can be opened at the shoulders for ease of use.

3. The vest has a slit on each side, making it comfortable to sit down.

4. Can be adjusted at both sides and at the back.

5. As a safety precaution, the adjustable side and back draw strings are situated in concealed pockets.

## Product service life

The product has an expected service life of 2 years.

This is based on correct use, cleaning, washing, service and maintenance.

Products used beyond the aforementioned service life should be assessed for further use.

Responsibility for using the product beyond the stated service life rests with the user.

## Cleaning

The Protac MyFit® must be washed in the wash bag supplied. The regularity and need for washing depends on use. See below for washing and cleaning instructions.

- The vest can be machine washed in a min. 7 kg capacity washing machine with washing detergent with no bleaching agents - see also the washing instructions in the vest.
- The enclosed wash bag must be used. Spinning at low speed.

To remove spots using a spot remover, always try first on an area which is not visible to test for damage or discolouring. Protac does not recommend the use of spot remover. Protac cannot be held liable for incorrect washing and maintenance.

## Maintenance

We recommend airing the vest regularly.

## Transport and storage

The vest is ready for use.

For transport, Protac MyFit® is packed in a carrying bag to protect the vest against dirt and damp.

We recommend storing the vest flat and covered to protect against dirt and damp.

## Disposal

Protac MyFit® can be divided into the following components for waste disposal purposes: Plastic and fabric.

The plastic balls can be delivered for recycling.

All parts of the vest can be disposed of as combustible waste.

Waste must be disposed of in accordance with applicable rules.

## Guarantee

2 year guarantee against manufacturing defects.

### The guarantee will be invalid in the event of the following:

- over-use
- misuse
- non-maintenance or incorrect maintenance

**Traceability:** Production month, year and batch number are marked on the washing label.

## Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product

**Washing bowl:** Maximum washing temperature (see care label)

**Warning:**  
See under **Safety**

**Triangle:**  
Do not bleach

User information

**Square with a circle:**  
tumble drying possible  
(see care label)

For indoor use

**Iron:**  
Do not iron

Manufacturer

**Circle:**  
Do not dry clean

Batch number

## Technical specifications

**Standards:** Comply with general requirements for aids acc. to the following harmonized standards:

**Clinical evaluation:** EN 12182:2012

**Risk analysis:** DS/EN ISO 14971

## Service information

No other balls can be used in a Protac MyFit® than those used by the manufacturer.

## Guide

- to daily inspection of the Protact MyFit® before use

GB

No.	Check	Remarks
1	<ul style="list-style-type: none"><li>All parts are in place, no loose parts</li></ul>	
2	<ul style="list-style-type: none"><li>No sign of holes, damage to the textile or seams</li></ul>	

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### Protac A/S