## Protac SenSit® Nature Puff

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GB - Directions for use

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SE - Bruksanvisning

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Medical Device

[www.protac.dk](http://www.protac.dk)

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## Protac SenSit® Nature Puff

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# GB - Directions for use

## Protac SenSit® Nature Puff

You have chosen a Protac quality product. Protac guarantees that you have purchased an original Danish design product developed in partnership with professionals.

Since 1994, Protac has developed, manufactured and sold sensory stimulating aids to the healthcare sectors in Denmark, Europe, Australia and New Zealand.

Protac products are developed from the theory of sensory integration and are well documented with respect to research, documentation and case descriptions.

Protac production process focuses on environment and safety. The fabric is certified according to Standard 100 by OEKO-TEX® nr. 1606038 Centexbel.

Protac plastic balls are Standard 100 by OEKO-TEX® nr. 2076-310 DTI.

GB

For more information, go to [www.protac.dk](http://www.protac.dk)

## Labelling

The products are labeled in accordance with Regulation (EU) 2017/745 and 93/42/EC Class I medical devices. CE- and care labels with information about the manufacturer, raw materials and washing instructions are placed inside the puff.

Symbol explanation on label: see the Key to symbols section.

## General precautions - Safety

- ⚠ • The Puff must not be used unsupervised by individuals who require special care.
- ⚠ • Must not be used by individuals with pressure ulcers on their calves and heels.
- ⚠ • The Puff must not be used as a stool, by individuals with bedsores.
- ⚠ • Take care not to trip over the footstool if the user is unsure of his/her balance.
- ⚠ • Protac SenSit® Nature Puff must not be placed or affixed to a platform with wheels.
- ⚠ • Avoid smoking when sitting on the stool. Lost embers or cigarettes can cause burns.
- ⚠ • Must not be used near an open flame.
- ⚠ • Do not leave a person sitting on the stool in direct sunlight for long periods of time.
- ⚠ • The user must use sunscreen if Protac SenSit® Nature Puff is used in the sun.
- ⚠ • Max. weight 150 kg

## Purpose and function of the product

All Protac products are developed from the theory of sensory integration. The theory originates from the USA and was developed by occupational therapist and psychologist A. Jean Ayres back in the 1970s. The theory is based on how the brain processes sensory impressions and how the different sensory stimuli can be influenced through weight and pressure on the tactile sense (sense of touch) in the skin and the proprioceptive sense (sense of movement) in muscles and joints. The pressure, movement and weight of the balls stimulate these particular senses.

The Protac Sensit® Nature Puff was developed as an aid for individuals with functional impairments.

The Protac Sensit® Nature Puff is suitable for children, youths, adults and the elderly.

- Physical and psychological unrest
- ADHD and autism
- Psychological disorders such as dementia, schizophrenia, depression, anxiety, anorexia etc.
- Developmental disorders, sensory disruptions, deaf-blindness etc.
- Stress
- Balance problems

For people with motor unrest and autism, Protac Sensit® Nature Puff is used as a stool where you get the sensory stimulation which create security through the seat with the balls.

People with psychiatric disorder often have motor and psychological unrest. Protac Sensit® Nature Puff can be used as a stool to create peace and security for the user. The pressure of the balls also gives a better body sensation and increased well-being.

Protac Sensit® Nature Puff is used for people with e.g. developmental disorders, sensory disorders, deafblindness to create increased sensory stimulation to the body through the pressure of the balls.

Protac Sensit® Nature Puff is used for people with stress to provide calm, security and a better body awareness.

Protac SenSit® Nature Puff can e.g. be used in outdoor spaces, gardens and parks, in institutions, residences, day care centers and hospitals.

In schools, Protac SenSit® Nature Puff can e.g. used in the schoolyard or other outdoor spaces. In private homes, Protac SenSit® Nature Puff can be used e.g. on the terrace, in the conservatory or in the garden.

## Using the product

The Protac SenSit® Nature can be used as a foot/leg rest, together with the Protac SenSit® Nature.

GB

The Protac SenSit® Nature can be used as a stool.

## Protac SenSit® Nature Puff is delivered assembled and ready for use

### Product lifetime

The product's expected lifetime is three years.

The product's lifetime is based on correct use, cleaning, washing, service and maintenance. Products that are used in excess of the above lifetime must be assessed for future use. The user is responsible for continuously assessing whether the product can be used in excess of the lifetime stated.

### Cleaning

The cover for Protac SenSit® Nature Puff can be washed. The need and frequency of washing depend on the use of the chair. Follow the instructions below when washing and cleaning the chair.

- The cover can be washed in a washing machine, min. 7 kg, using a detergent without bleach and perfume. See the washing instructions inside the cover.
- The innerbag with balls and granulates cannot be washed. See the washing instruction on the innerbag.

### Disassembly before washing:

1. Remove the inner bag from the puff.
2. Zip the zipper before washing the cover.

## Assembly after washing:

1. Zip up the zippers.
2. The inner bag is placed in the cover, the zipper is zipped.

Before using any stain removers, do a test on a non-visible area to make sure that it does not damage or change the colour of the fabric. Protac does not recommend the use of stain removers.

## Maintenance

Inspect the Protac SenSit® Nature Puff for defects and deficiencies prior to each use. Inspect the all seams, zips, fabric, straps, balls and general condition once a year. If serious defects or deficiencies are discovered, the Protac SenSit® Nature Puff must not be used until the defects have been repaired and safety has been restored.

The Protac SenSit® Nature Puff can be wiped with a well-wrung cloth soaked in warm soapy water. Can be wiped off with alcohol.

- ⚠ • Protac SenSit® Nature Puff must not be used in temperatures below 0 degrees.
  - After heavy rain, turn the stool upside down and wipe the bottom with a dry cloth.

Protac accepts no liability for damages resulting from incorrect washing and maintenance.

## Transport and storage

During transport, the Protac SenSit® Nature Puff is packed in a cardboard box, which protects it against dirt and moisture.

If the stool is used together with the Protac SenSit® Nature chair it can, when not in use, be placed on the chair, so that when the chair's raincover is pulled over, both chair and stool are protected against, among other things, bird droppings, sun and heavy rain.

## Disposal

Protac SenSit® Nature Puff can be divided into the following components for waste disposal purposes: Plastic and fabric.

The plastic balls can be disposed of as hard plastic.

All parts of the chair can be disposed of as combustible waste.

Waste must be disposed of in accordance with applicable rules.

## Warranty

There is a two-year warranty against manufacturer's defects.

## The warranty will be voided in the following cases:

- Overload of Protac SenSit® Nature Puff
- Harsh use of Protac SenSit® Nature Puff
- Lack of or incorrect maintenance of Protac SenSit® Nature

**Traceability:** Production month, year and batch number are marked on the washing label.

## Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product.

Washing bowl: Maximum washing temperature (see care label)

Triangle:  
Do not bleach

Square with a circle:  
tumble drying possible  
(see care label)

Iron:  
Do not iron

Circle:  
Do not dry clean

Warning:  
See under Safety

User information

For Outdoor use

CE marked according to  
(EU) 2017/745 and 93/42/EC  
Class I medical devices

Manufacturer

Batch number

GB

## Spare parts

- Cover
- Innerbag with balls and granulate.

## Technical specifications

### Dimensions:

Protac SenSit® Nature Puff

Height: 42 cm.

Weight: approx. 2,5 kg.

**Standards:** Comply with general requirements for aids acc. to the following harmonized standards:

**Clinical evaluation:** EN 12182:2012

**Risk analysis:** EN ISO 14971:2019

**Quality management systems:** EN ISO 13485:2016

**Watercolumn for outer fabric:** 10.000 mm - EN ISO 811: 2018

## Service information

Only the balls used by the manufacturer may be used in the Protac SenSit® Nature.

## Guide

- to daily pre-use inspection of the Protac SenSit® Nature Puff.

1. Zipper is closed
2. No sign of holes or damage to the fabric and seams

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Protac A/S