PROTAC®

INTEGRATING SENSES

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GB - Directions for use

Protac Incontinence Cover™

Protac Incontinence Cover™ with MESH

You have selected a quality Protac product. Protac guarantees that you have purchased an original, Danish designed product developed in consultation with experts. Protac has developed, produced and sold sensory-stimulating aids to the healthcare sector in Denmark, the rest of Europe, Australia and New Zealand since 1994. Protac products are developed on the basis of theories on sensory integration, and are supported by extensive research, documentation and case studies. Protac focuses strongly on protection of the environment and safety during its production process. The fabric is certified according to STANDARD 100 by OEKO-TEX® no 60616412 Centexbel.

For more details, go to www.protac.dk

GB

Labelling

The products are labeled in accordance with Regulation (EU) 2017/745 Class I medical devices. The label with details of manufacturer, raw materials and washing instructions is sewn inside the cover to avoid irritating the user.

Key to label symbols: please refer to Key to symbols

General precautions - Safety

- ⚠ • We recommend that the opening of the incontinence cover is at the opposite end to that of the duvet cover (not applicable for incontinence cover with mesh top)
- ⚠ • The same risk precautions apply as for the use of a plastic bag:
 - Protac Incontinence cover is a dangerous toy for children
 - A Protac Ball Blanket® used with an incontinence cover must not be used for children without supervision.
 - A Protac Ball Blanket® with incontinence cover must not be used for patients unable to care for themselves without supervision.
 - A Protac Ball Blanket® used with an incontinence cover must not be pulled over the head.
- ⚠ • We recommend using the incontinence cover version with a mesh top (a breathable, net panel)
 - ⚠ for users unable to care for themselves.
 - Must not be used near an open flame.

Product purpose and areas of use

The Protac Incontinence Cover™ was developed as an accessory for the Protac Ball Blanket®.

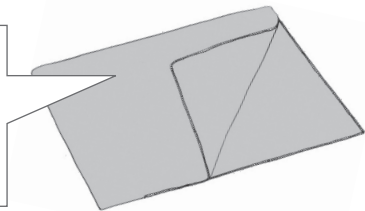
Children, young and old people can benefit from the cover.

The Protac Incontinence Cover™ is used to protect the Protac Ball Blanket® or other duvets in the event of patient incontinence.

Using the product

The cover is intended to fit over the Protac Ball Blanket®. An ordinary duvet cover should be used over this.

The cover is easy to fit, with zips on two sides.

GB

Product service life

The product has an expected service life of 5 years.

This is based on correct use, cleaning, washing, service and maintenance.

Products used beyond the aforementioned service life should be assessed for further use.

Responsibility for using the product beyond the stated service life rests with the user.

Cleaning

All Protac Incontinence Covers can be washed. The regularity and need for washing depends on use. See below for washing and cleaning instructions.

Protac Incontinence Cover

- Incontinence cover, item no. 300-302 and 300-304 can be washed in a min. 6 kg capacity washing machine with detergent with no bleaching agents or perfume - see also the washing instructions in the cover.

⚠ • Spot remover should not be used to avoid damage to the PU coating.

Protac cannot be held liable for incorrect washing and maintenance.

Maintenance

We recommend washing the cover regularly. Use of a laundry bag is recommended.

Transport and storage

The Protac Incontinence Cover™ is supplied in a sealed plastic bag.

The cover is ready for use.

If you are not using the cover, store it carefully in its bag to protect from dirt and damp.

Disposal

The cover can be disposed of as combustible refuse.

Always dispose of refuse according to local rules.

Guarantee

2 year guarantee against manufacturing defects.

The guarantee will be invalid in the event of the following:

- over-use
- misuse
- non-maintenance or incorrect maintenance

Traceability: Production month, year and batch number are marked on the CE- and care label.

Key to symbols

Example only. Always follow the recommended washing instructions on the label found inside the product

Washing bowl: Maximum washing temperature (see care label)

Warning:
See under Safety

Triangle:
Do not bleach

User information

Square with a circle:
tumble drying possible
(see care label)

For indoor use

Iron:
Do not iron

CE marked according to
(EU) 2017/745 for Class I
medical devices

Circle:
Do not dry clean

Manufacturer

Batch number

Technical specifications

Dimensions, Protac Incontinence Cover™:

Protac Incontinence Cover™, Adult:	Width: 140 cm	Length: 200/220 cm
Protac Incontinence Cover™ with MESH at the top, adult:	Width: 140 cm	Length: 200/220 cm
Protac Incontinence Cover™, Junior:	Width: 100 cm	Length: 140 cm
Protac Incontinence Cover™ with MESH at the top Junior:	Width: 100 cm	Length: 140 cm

Standards: Comply with general requirements for aids acc. to the following harmonized standards:

Clinical evaluation: EN 12182:2012

Risk analysis: DS/EN ISO 14971

Fire test:

Style 300-302 comply with ISO 12952 part 1 & 2 : 2010.

Model 300-304 is made of polyester with PU coating and polyester MESH net on the top - this model is not flame retardant.

Guide

- to daily inspection of the Protact Incontinence Cover™ before use

1. All parts are in place, no loose parts

GB

2. No sign of holes, damage to the textile or seams

Protac A/S
Niels Bohrs Vej 31 D, Stilling
DK-8660 Skanderborg
Tel.: +45 86 19 41 03
e-mail: protac@protac.dk

Distributor list: Please refer to: www.protac.dk/kontakt

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Protac Incontinence Cover™

Protac Incontinence Cover™ with MESH

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