## Protac SenSack®

DK - Brugsvejledning  
GB - Directions for use  
DE - Gebrauchsanweisung  
FI - Käyttöohje  
FR - Mode d'emploi  
NL - Gebruiksaanwijzing  
SE - Bruksanvisning  
ES - Instrucciones de uso  
IT - Istruzioni

Medical device

[www.protac.dk](http://www.protac.dk)

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## Protac SenSack®

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# GB - Directions for use

## Protac SenSack®

You have selected a quality Protac product. Protac guarantees that you have purchased an original, Danish designed product developed in cooperation with experts. Protac has developed, produced and sold sensory stimulating aids to the healthcare sector in Denmark, the rest of Europe, Australia, New Zealand, Japan and South Korea. Protac products are developed on the basis of theories on sensory integration, and are supported by extensive research, documentation and case studies. Protac focuses strongly on protection of the environment and safety during its production process. Protac plastic balls are certified according to Standard 100 by OEKO-TEX® nr. 2076-310 DTI. The fabric is certified according to STANDARD 100 by OEKO-TEX®. nr. 1407028 Centexbel. For more details, go to [www.protac.dk](http://www.protac.dk)

GB

## Labelling

The products are marked in accordance with (EU) 2017/745 for medical devices class I. Label with information about manufacturer, raw materials and washing instructions is attached inside Protac SenSack®.

Key to label symbols: please refer to Key to symbols

## General precautions - Safety

- ⚠ • Not for unsupervised use by users unable to care for themselves.
- ⚠ • Not for unsupervised use by users with low lungcapacity.
- ⚠ • Not to be used as a means of restraint.
- ⚠ • Be aware of the user's reaction to overstimulation if the user feels uncomfortable lying in the sleeping bag.
- ⚠ • Avoid smoking while lying in the sleeping bag. Lost embers or cigarettes can cause burns.
- ⚠ • Never leave a computer or other electrical appliances switched on in a bed with a Protac SenSack®, as they emit heat which can be harmful to the blanket, or in extreme circumstances, cause the appliance to ignite, due to a build-up of heat.
- ⚠ • Must not be used near an open fire.
- ⚠ • Make sure that the user does not pull the sleeping bag over his head
- ⚠ • Be aware of breathing problem if the user is using the sleeping bag cover.

## Incidents

- Incidents related to the use of Protac SenSack® must be reported to the manufacturer
- Serious incidents related to the use of Protac SenSack® must be reported to the manufacturer and the competent authority of the country where the incident takes place.

## Product purpose and areas of use

Sensory integration is the idea behind the development of all Protac products. The theory originated in the 1970s in the USA, and was developed by A. Jean Ayres, an occupational therapist and psychologist. The theory is based on how the brain processes sensory impressions, and how the various sensory stimuli can be influenced via applying weight and pressure to the tactile sense (the sense of touch) in the skin and the proprioceptive sense (the sense of movement) in muscles and joints. The pressure, movement and weight of the balls act on these senses in particular.

The Protac SenSack® was developed to help sufferers of function impairment. Children, young people and elderly can benefit from the Protac SenSack® :

GB

- Brain damage, including spasticity, Huntington's Chorea, Parkinson's Disease and other ailments
- ADHD and autism
- Psychiatric diseases, such as dementia, schizophrenia, depression, anxiety, anorexia etc.
- Development problems, sensory problems, deaf/blind, etc.
- Stress

The Protac SenSack® can be used for patients with brain damage resulting in spasticity. The weight and pressure of the balls help suppress spasticity, and calm the body of patients with brain damage. Furthermore, the patient benefits from enhanced body awareness.

The Protac SenSack® is used by ADHD and autism sufferers to sleep with, and as a source of comfort during the daytime. The pressure and weight of the balls impart sensory stimuli which help define the body, giving comfort and calm. ADHD sufferers often have a problem winding down at night, and the Protac Ball Blanket® gives the body a signal to relax through its pressure and weight.

Psychiatric patients often suffer from muscular and psychological unrest. The Protac SenSack® is used to create calm and comfort for the user. The weight and pressure of the balls also give a better feeling of body awareness and enhanced wellness. The Protac SenSack® is used to sleep with, and to provide comfort during the daytime.

The Protac SenSack® is used for sufferers of a range of development problems, sensory problems, deaf/blindness and to create enhanced sensory stimuli in the body.

Protac SenSack® is used for people with sleep disorders. Particularly sensitive users and especially children with ADHD often have difficulty falling asleep. Likewise, many people with disabilities have sleep disorders.

The Protac SenSack® is used for sufferers of stress to provide calmness, comfort and better body awareness. It is used to sleep with, and to provide comfort during the daytime.

Protac SenSack® can be used in institutions, residences, day care centers, hospitals, living rooms and private spaces. In private homes, Protac SenSack® can be used e.g. in the user's room and in the living room. It is easy to take the sleeping bag with you on weekends or at "play appointments". It is designed to look like a regular sleeping bag. It is delivered in a transport bag. A cover has been made as an accessory so that the sleeping bag can be used outside for outdoor rehabilitation.

## Using the product

Protac SenSack® is used for sleeping. Wrap it tightly to the body so that you feel the balls clearly

It is a good start to try to sit with the sleeping bag or lie in the sleeping bag in a chair or couch, to feel the weight and the balls, which will be a new feeling

GB

The top of the sleeping bag is designed so that it can either be used as a hood and thus contribute to extra security or it can be tilted around and instead act as a small pillow.

The sleeping bag has zippers on both sides, which can be opened either from the top or the bottom, and it is thus possible to get air to the feet.

**The Protac SenSack® is ready for use.**

## Product service life

The product has an expected service life of 5 years.

This is based on correct use, cleaning, washing, service and maintenance.

Products used beyond the aforementioned service life should be assessed for further use.

Responsibility for using the product beyond the stated service life rests with the user.

## Cleaning

Protac SenSack® can be washed. The regularity and need for washing depends on use.

See below for washing and cleaning instructions.

GB

- The sleeping bag is divided in two and each part is washed in a washing machine with detergent without bleach and perfume. Rinse aid must not be used. Spinning at low speed - see also washing instructions in the sleeping bag.
- Size S: Each part can be washed in a washing machine with a min. capacity of 7 kg
- Sizes M and L: The lower part can be washed in a washing machine with min. capacity of 7 kg. The upper part must be washed in a washing machine with min. 16 kg capacity
- Each part to be dried in a tumble dryer together with tennis balls

To remove spots using a spot remover, always try first on an area which is not visible to test for damage or discolouring. Protac does not recommend the use of spot remover.

Protac cannot be held liable for incorrect washing and maintenance.

## Maintenance

We recommend hanging the Protac SenSack® out for airing at regular basis.

## Transport and storage

Protac SenSack® is delivered packed in a transport bag with carrying strap and in a carton, which protects the sleeping bag from dirt and moisture. Protac SenSack® can advantageously be transported in the transport bag, so it is easy to handle. If the sleeping is not being used, it should be stored securely in the transport bag so that it is protected from dirt and moisture.

## Disposal

The Protac SenSack® can be broken down into the following components for disposal:  
Plastic and textile.

The plastic balls can be disposed of as hard plastic.

All parts of the chair can be disposed of as combustible waste.

Waste must be disposed of in accordance with applicable rules.

## Guarantee

2 year guarantee against manufacturing defects.

### The guarantee will be invalid in the event of the following:

- overloading
- misuse
- non-maintenance or incorrect maintenance

## Accessories

Protac SenSack® Cover

As long as the sleeping bag is taken outdoors, a cover bag should be used.

**Traceability:** Production month, year and batch number are marked on the washing label.  
These must not be removed from the product.

## Key to symbols

Example only. Always follow the recommended care instructions on the label found in the product

Washing bowl: Maximum washing temperature (see care label)

Triangle:  
Do not bleach

Square with a circle:  
tumble drying possible  
(see care label)

Iron:  
Do not iron

Circle:  
Do not dry clean

Warning:  
See under Safety

User information

For indoor use

CE marked according to  
(EU) 2017/745 Class I  
medical devices

Manufacturer

Batch number

## Technical specifications

Dimensions:

Protac SenSack® S:	length: 139 cm	weight: ca. 4 kg
Protac SenSack® M:	length: 196 cm	weight: ca. 5,5 kg
Protac SenSack® L:	length: 216 cm	weight: ca. 7,5 kg

**Patent:** Protac SenSack® - Patent pending

**Standards:** Comply with general requirements for aids acc. to the following harmonized standards.

**EN ISO 13485:2016/A11:2021:** Medical Devices – Quality management systems

**EN ISO 14971:2019/A11:2021:** Medical devices - Application of risk management to medical devices

**DS/EN ISO 21856:2022 :** Medical devices - Assistive products - General requirements and test methods.

GB

## Service information

No other balls can be used in a Protac SenSack® than those used by the manufacturer.

## Guide

- to daily inspection of the blanket before use

- 1 • All parts are in place, no loose parts
- 2 • No sign of holes, damage to the textile or seams
- 2 • Zippers are intact

Protac A/S  
Niels Bohrs Vej 31 D, Stilling  
DK-8660 Skanderborg  
Denmark  
Tel.: +45 86 19 41 03  
e-mail: protac@protac.dk

Distributor list: Please refer to: [www.protac.dk/kontakt](http://www.protac.dk/kontakt)

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© nr. 2076-310 DTI. The fabric is certified according to STANDARD 100 by OEKO-TEX®. nr. 1407028 Centexbel.

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